

IN BRIEF: Lightwriter SL35 (DECtalk®) and Lightwriter SL38 (Acapela)

Press On/C to turn on

Full details of operation in the Manual LIT403RF

You should also have a QUICKREF LIT501QK

MEM

- The MEM-key has many uses:

MEM
MEM-MEM
MEM-MEM-MEM

this level is for recalling memories.
this level is for saving memories.
this level is for executing functions.

- For example:

MEM-1 is 'AND'
MEM-P is 'WOULD YOU PLEASE'
MEM-5 is a list of memories
internal device.
MEM-X is 'empty'
MEM-zero is 'louder'
MEM-MEM
MEM-MEM-MEM-H

overwrite it, if you want.
overwrite it, if you want.
all memories are backed up to a secure

you can store up to 36 'Direct-Memories', plus Abbreviation-Expansions.
you could overwrite it, as it is always available in setups (see below).
save whatever is on the screen.
enter the help-system



SETUPS

- You can adjust the Lightwriter in many ways, from the everyday to the specialised. Examples:

Everyday


Volume	select the speech-volume.
Voice	select Male or Female.
Speak: Word	"speak each time I press Space".
Speak: Replay	"don't speak until I press Replay".

Specialised

Port	configure the serial-ports (2 on 35, 1 on SL38).
Pitch	select the pitch of the voice.
Speak: Key	spoken confirmation of keys (for learning-disabilities).
Speak: Announce	announce each key when pressed with no action (for low-vision).

USE ALL DAY, CHARGE EACH NIGHT

KEYS

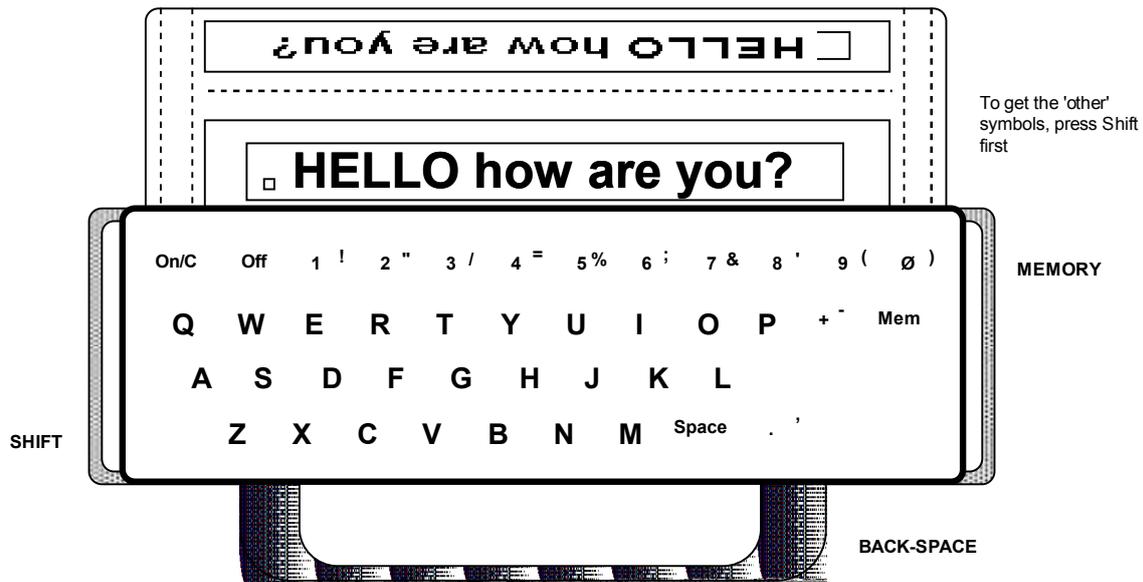

- Press **Replay** to repeat the message. **Shift** then **Replay** to spell the last word.
- Press **Shift** then **Buzz** for an alarm. You can alter the sounds.

Language-change, where purchased (SL38 only)

- Lightwriter SL38 (Acapela speech) can be fitted with more than one language.
- To switch from one language to another, press these keys:
 - MEM** then **MEM** then **MEM** then **L** then **1** then [confirm it] to load: American-English
 - MEM** then **MEM** then **MEM** then **L** then **2** then [confirm it] to load: Spanish
 - MEM** then **MEM** then **MEM** then **L** then **3** then [confirm it] to load: Italian
 - MEM** then **MEM** then **MEM** then **L** then **4** then [confirm it] to load: French
- If you load a language that you have not purchased, the Lightwriter will not speak.
- Having loaded a new language, you may need to change some setups.
- You can save these changes permanently. The sequence of events is:

Change the setups to how you would like them (eg: a Female voice)

Clear the display, type the word **BARNWELL**

Save this new "profile" - **MEM** then **MEM** then **MEM** then **S** then the desired number, then [confirm it]

From now on, **MEM MEM MEM L** then the number will load the language with your preferences

Literature Part No: LIT512BB 31 May 2006 (DynaVox)

Designed and manufactured in the UK by Toby Churchill:

Distributed by Spectronics:

TOBY CHURCHILL Limited
Norman Way Industrial Estate
OVER,
Cambridge, CB4 5QE,
United Kingdom
Ph: (+44) 1223 576117
www.toby-churchill.com

Spectronics Australia
PO Box 88
Rochedale QLD 4123
Australia
Ph: (07) 3808 6108
Email: mail@spectronicsinoz.com
Web: www.spectronicsinoz.com

Spectronics New Zealand
PO Box 20 1186
Auckland Airport, MANUKAU CITY 2150
New Zealand
Ph: (09) 275 5744
Email: mail@spectronics.co.nz
Web: www.spectronic.co.nz

